



Sling comfort is everything to do with the correct fit!

An ill-fitting sling leads to discomfort as well as impairing performance.

Our unique anthropometric sizing system is based on the science of universal Vitruvian Man body ratios. Therefore, unlike using generalist sizing templates which can vary between manufacturers, Care & Independence sling sizing is in sync with *individual personal measurements*.

This means that our slings will deliver the correct fit, greatest comfort and optimal performance without the guesswork.

HOW TO MEASURE

Height

For accuracy, measure a persons height whilst they are lying flat on a bed. If you measure nape of neck to coccyx, knee to hip, etc., you will accumulate inaccuracies and almost always result in a person measuring taller than they really are, potentially leading to a sling that is too big for them.

Girth


The second measurement you will need is the girth. This can be done whilst the person is seated or lying down. Measure right around the widest part of the body (usually hips on women and chest just under the arms on men and children).

FOXY SLING STYLE

If a User's girth size is too large to match the height column in the main Hybrid chart, a Foxysized sling may offer a better fit.

The Hybrid Foxy sling variant is specifically for people with broad hips and larger thighs who find that the leg supports on other slings are not long enough to go around their hips and between their thighs comfortably.

SLING SIZE GUIDE FOR COMFORT VALUE™

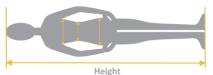

GirthMeasure the widest part of the body

Measure whilst lying flat

| 2 | | Height cm | Girth cm | Max.Load kg | Size | Binding Colour |
|---------|--------|-----------|----------|-------------|------|----------------|
| STANDAR | Unisex | 130-150 | 65-80 | 230 | XXS | Orange |
| | | 135-160 | 70-85 | 230 | XS | White |
| | | 145-170 | 75-90 | 230 | S | Red |
| | | 155-180 | 85-100 | 230 | M | Yellow |
| | | 160-190 | 95-115 | 230 | L | Green |
| | | 160-190 | 105-130 | 230 | XL | Lilac |

| _ | | | | | | |
|--------|----------------------|-----------|----------|-------------|------|----------------|
| \Box | | Height cm | Girth cm | Max.Load kg | Size | Binding Colour |
| HYBRID | Paediatric Unisex | 75-95 | 45-55 | 230 | 1 | White |
| | | 95-110 | 50-60 | 230 | 2 | Red |
| | | 110-130 | 55-70 | 230 | 3 | Yellow |
| | | 130-150 | 65-80 | 230 | 4 | Orange |
| | Adult Unisex | 135-160 | 70-85 | 230 | 5 | White |
| | | 145-170 | 75-90 | 230 | 6 | Red |
| | | 155-180 | 85-100 | 230 | 7 | Yellow |
| | | 160-190 | 95-115 | 230 | 8 | Green |
| | | 160-190 | 105-130 | 230 | 9 | Lilac |
| | | 160-190 | 125-150 | 350 | 10 | White |
| | | 160-190 | 145-175 | 350 | 11 | Red |
| | | 160-190 | 170-200 | 350 | 12 | Yellow |

| ≿ | | Height cm | Girth cm | Max.Load kg | Size | Binding Colour |
|---|--------------|-----------|----------|-------------|------|----------------|
| 9 | | 130-155 | 85-100 | 230 | 5 | White |
| | | 135-160 | 95-110 | 230 | 6 | Red |
| 쫉 | | 140-165 | 105-125 | 230 | 7 | Yellow |
| 물 | Adult Unisex | 140-165 | 120-140 | 230 | 8 | Green |
| ᄑ | | 140-165 | 135-155 | 230 | 9 | Lilac |